


the power of

elōs

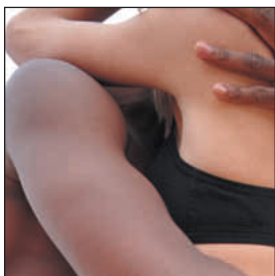

Skin
Rejuvenation

Hair Removal
FAQs

– Frequently Asked Questions

What is Elos?

ELOS stands for **electro optical synergy** and combines bipolar conducted electric current with optical (light) energy.

What is the Elos Advantage?

Elos technology provides skin rejuvenation and hair removal for all skin and hair types. Compared to earlier IPL technology, there is less blistering, scabbing, crusting and hyper or hypopigmentation, reduced redness and inflammation and less pain.

What can Elos technology be used for?

Elos technology can be used to:

- Rejuvenate skin (Elos FotoFacial™)
- Repair sun damaged skin
- Treat pigmented lesions, including acne scars
- Treat vascular lesions
- Improve skin texture – wrinkles and fine lines
- Remove unwanted hair on all skin types and hair colours - even white and grey

Why is Elos technology more effective than standard IPL?

By emitting the optical pulse and electric pulse at the same time, the net effect is to force the electric energy to stream preferentially to the regions of the dermis pre-heated by the absorbed light, producing a greater and more focused thermal effect than a single pulse of high energy light.

With less intense light energy there are less burning risks and the synergistic coupling of energies means that each pulse can be delivered for a longer period of time than standard IPL – yielding improved results.

Is treatment with Elos technology safe?

Treatment is relatively safe for the skin. There are many advances in the new Elos technology that make it one of the safest skin rejuvenation and hair removal systems available, for all skin types and hair colours. There are no long-term health hazards and both light and electric energy have been used for decades in medicine, surgery and aesthetics without adversity.

Elos Technology – FAQs For Hair Removal

How does the Elos technology get rid of hair?

Gentle light and electric energies combine to damage the hair follicle. This disables the follicle from growing hair, which means that gradually over several treatments, there is a long-term reduction in the amount of hair in that area.

What kind of results can be expected from Elos hair removal treatment?

Each treatment will result in less hair coming back, as well as hair growing progressively lighter in colour, becoming finer and also slowing regrowth. It takes more than one treatment to affect all the follicles growing in an area. Most people achieve satisfactory clearance after 4-8 treatments, however, individual results vary depending on medical and genetic factors. Lighter coloured hair may require more treatments than darker coloured hair.

How frequently do hair removal treatments with Elos technology need to be performed?

Most people achieve satisfactory clearance after 4-8 treatments, which are usually performed every 1-2 months. This depends on individual factors that the practitioner will discuss during consultation.

How does hair removal with Elos technology feel?

The sensation is often described as a 'hot pinch' or 'snapping' feeling. The sensation only lasts for a fraction of a second and warmth and/or a tingling sensation is experienced for a short time afterward. Most people tolerate treatment without any topical anaesthetic, but individuals who are more sensitive may prefer to have their skin numbed before treatment.

What happens after each Elos hair removal treatment?

Temporary pinkness in the skin lasts a very short time and most people return to work or normal activities immediately after treatment. The treated area will 'shed' some hairs over the next 3-4 weeks and there may be a short period of irregular hairlessness in the area until the conclusion of treatments.

Who is suitable for Elos hair removal treatment?

Anyone seeking a long-term solution to shaving, waxing, plucking or electrolysis. There are a few medical conditions that contraindicate people from treatment that the practitioner will discuss during consultation.

Elos Technology – FAQs For Skin Rejuvenation

How does the Elos technology rejuvenate the skin?

The gentle light and electric energies break up the pigment and/or blood in lesions and stimulates collagen restructuring and/or production, restoring skin to a more youthful state. Brown spots (pigmented imperfections) from sun damage and aging, red blotchiness from spider vessels or rosacea (vascular imperfections) and rough skin texture will improve over a treatment series.

What kind of results can be expected from an Elos FotoFacial™ (facial rejuvenation) treatment?

Each Elos FotoFacial™ treatment will result in gradual lightening and improvement of pigmented and vascular imperfections. Textural improvements are appreciated towards the end of the treatment series. Here are some of the changes that can be expected:

- With a series of five treatments people with ruddy complexions should see an improvement some-where between 60 and 70%
- Broken capillaries should resolve about 75%
- Sunspots, brown spots and freckles will improve anywhere from 80 to 90%
- Fine wrinkles around the eyes and mouth area improve around 40 to 60%
- Deeper wrinkles will improve anywhere from 20 to 40%

How frequently do Elos FotoFacial™ treatments need to be performed?

Treatments are repeated every 2-4 weeks for an average of five treatments.

How does the Elos FotoFacial™ treatment feel?

The sensation is often described as a 'hot pinch' or 'snapping' feeling. The sensation only lasts for a fraction of a second and warmth and/or a tingling sensation is experienced for a short time afterward. Most people tolerate treatment without any topical anaesthetic, but individuals who are more sensitive may prefer to have skin numbed before treatment.

What happens after each Elos FotoFacial™ treatment?

Temporary pinkness in the skin lasts a very short time, and most people return to work or normal activities immediately after treatment. The improvement in pigmented and vascular irregularities can often be appreciated before the next treatment a month later. Brown spots temporarily go darker before they lighten.

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
Who can experience the benefits of an Elos FotoFacial™ skin rejuvenation treatment?

Anyone who wants to improve the signs of sun damage and ageing in the skin, such as pigmented and vascular irregularities, uneven or rough skin texture. Skin rejuvenation can be performed on face, neck, chest, hands and arms.

elōs



Make the superior choice
– choose elos



Call now for more information or your nearest location

1800 888 983

I N C I S I O N PTY LTD

Syneron

This brochure is intended for information purposes only. Consult your doctor about what procedure is most suitable for you.